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## Directions for Use – MosQuit® – Electronic device against insect bites

**Medical explanation of how your MosQuit® works** •While biting, insects inject an aluminous secretion into the puncture. It prevents the clotting of blood and activates the endogenous defence of the human body. The body releases histamine. This antitoxin causes itching and pain. Rubbing or scratching the puncture may increase itching and can cause inflammation. Concentrated heat from a temperature of 47°C, can decompose the insect poison. So the results of the insect bite such as itching, swelling and the hazard of infection can be gently eliminated within a few seconds.

**When to use your MosQuit®** •Your MosQuit® insect bite healer can successfully prevent or considerably alleviate itching, swelling and the hazard of infection as a result of midge, mosquito and horsefly bites in up to 95 % of the cases.

**How to use your MosQuit®** •For the optimum effectiveness of your MosQuit®, you should use the device immediately after you have been bitten. Later application, however, can also effectively combat the negative consequences of an insect bite. The heating surface of your insect bite healer is located under the nose of the mouse. Place the heating surface exactly over the puncture. Keep your insect bite healer slightly pressed on during the time of application. Press one of the two buttons (mouse eyes). The unit heats up. After reaching the operation temperature during the treatment duration a signal is audible each ½ second. Press now the heating surface on the puncture. The treatment time can be defined via counting. It should take approx. 5 seconds or 3 seconds for children and persons which have a sensitive skin. You can stop the treatment at every time you like if you withdraw the MosQuit® from the puncture or let off the button. Therefore the MosQuit® does not need an automatic switch off.

**Frequency and time of application** •If possible, use your MosQuit® immediately after you have been bitten. With this unit you can do several applications directly one after the other for example at different parts of the skin!

**Side effects** •For proper usage, no side effects are known. There will be no risk of tissue damages such as burns, necrosis (cellular changes) or systemic effects when the device is used on sound and normal skin, however, an unpleasant heat feeling can be the result. For this reason, you should really apply your insect bite healer to insect bites only. Children and adults who have a sensitive skin should always select the short heating period. The heating surface consists of a gold plates iron/nickel composition, which is able to be allergenic if allergic persons contact it.

**Contraindications** •Your MosQuit® is no children's toy and not suitable for application to children below three years. Due to the heat it produces, do not use your MosQuit® within the delicate areas of your eyes and lips and on mucous membranes. Also, refrain from treating pre-affected skin or open wounds. Your MosQuit® is no medical emergency measure in case of well-known or acute allergy to insect poisons of horseflies, bees or wasps and can only avoid or considerably alleviate itching, swelling or the hazard of infection. In case of any unusual complaints after an insect bite, consult a doctor immediately. Nor does this MosQuit® substitute any prophylactic malaria treatment for stays abroad. Diabetics should not use the MosQuit®, because they may have a reduced pain sensitivity and would be able to inflict burns on themselves. It is not allowed to use the MosQuit® by in taking of alcohol, drugs or pain reducing pharmaceuticals. The MosQuit® should only be used on one person because of hygienically reasons.

**Warranty** •This device is warranted for a period of 24 months from the selling date. Repairs should only be carried out by the customer service department or by authorized dealers.

**Changing the battery** •Before a complain the battery has to be proven and has to be changed if it is necessary. The MosQuit® is powered by a conventional 9 V compound battery. It is not included in delivery. To put in/change the battery please open the battery compartment lid on the back of the device. Put in a new battery or replace the exhausted battery and slide back the lid over the battery compartment until it snaps in.

If the battery is too weak or empty a silent blip comes up which appears directly after pressing of one of the buttons of the MosQuit®.

**Cleaning and Unit Disposal** •Always remove the battery from the device before you clean it. Clean the device with a lightly damp cloth. Do not use any chemical cleaners or abrasive agents for cleaning. Make sure, that no water comes into the device. If this should ever happen, than only use the device once it is completely dry. Please dispose the unit according to the "Electro- and Electronic Altgeräte Verordnung 2002/96/EC-WEEE". Empty batteries should not be thrown into the domestic waste. You are legal obliged to dispose the batteries. If there are further inquiries please contact the communal public authority which is responsible for the disposal.

### Specifications:

Dimensions: 9 cm x 5 cm x 4 cm • Heating temperature: 47 °C - 54 °C  
Weight: 66 g incl. 9 V compound battery • Power supply: 9 V compound battery

This device has been tested in accordance with the legal regulations and standards for medical equipment and bears certification mark CE 0482 as per EC Directive 93/42/EEC and the Medical Products Law.

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Device classification type B

Do not dispose via domestic waste

Subject to technical modifications • Revision level 11040911

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### DAVITA® Medizinische Produkte GmbH & Co. KG