Gebrauchsanweisung User Manual

**Iontophorese-System
SUDORmed DE 20**

**Iontophoresis-System
SUDORmed DE 20**

Deutsch..... 2

English.....11

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Iontophorese-System

SUDORmed DE 20

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SUDORmed DE 20

Tables of Content

| | |
|---|-----------|
| Summary | 11 |
| Before operation | 11 |
| Safety instructions - side effects - contraindications | 12 |
| Further information for your safety | 12 |
| Intended usage, effectiveness and treatment plan | 12 |
| Components of the Iontophoresis-Systems SUDORmed DE 20 | 13 |
| Control unit of the SUDORmed DE 20 integrated in the case..... | 13 |
| Accessories included in delivery..... | 14 |
| Operation elements of the control unit | 14 |
| Starting up the Iontophoresis-Systems SUDORmed DE 20 | |
| and therapy performance | 15 |
| Summary of starting up and therapy performance | 15 |
| Starting up and therapy performance in detail. | 15 |
| Important, additional information for therapy performance | 16 |
| Additional information for treatment of hands, feet and underarms | 16 |
| Power supply | 17 |
| Control of battery capacitance..... | 17 |
| Charging by inserted rechargeable batteries..... | 17 |
| Care, maintenance and disposal | 17 |
| Guarantee | 18 |
| Technical specifications and scope of delivery | 18 |

Before operation

Dear customer,

We are happy you bought this product from DAVITA® and congratulate you on your choice.

You have bought a product which can be a contribution for the improvement of your quality of life for many years. In this user manual particularly the functions of the iontophoresis system are described in detail.

Safety instructions – side effects – contraindications

Up to now there are noted no dangerous side effects.

Attention:

It is not allowed to use the iontophoresis system regarding patients:

- ① with a heart pacemaker
- ① with an ICD (implanted cardiofibrillator)
- ① during pregnancy
- ① with metalliferous Intrauterine devices (Coil)
- ① with metal implants in the area of current flow (arms or legs)
- ① with large skin defects / wounds, which can not be masked with Vaseline
- ① with tumour diseases
- ① with thrombosis
- ① with strongly reduced sensibility of hands and feet (e.g. polyneuropathy)
- ① Children younger than 5 years

Warning:

- ① The electrodes must always be covered by the sponge cases provided. Otherwise burns could appear. Avoid the direct contact with the electrodes.
- ① Only use the non-conductive plastic trays provided.
- ① It is not allowed to use two units at the same time while treating one patient.
- ① Before the treatment, remove metallic jewellery (rings, bracelets etc). If metallic jewellery comes into contact with the water bath the electrical current concentration at these points can lead to light burns.

Further information for your safety

- ✚ Place the therapy unit on a flat surface.
- ✚ The device should be at room temperature when it is switched on.
- ✚ The unit should not be used near by short waves- and microwave therapy devices. Because it could result in making the iontophoresis device provide an incorrect electrical current. The distance should be minimum 1 meter.
- ✚ An effective current density form more than 2 mA/cm² at the electrodes makes an increased attention necessary. The electrodes should not bear on selectively because it can lead to very high current density and can trigger skin irritations at these places.
- ✚ Portable and mobile high frequency equipment (e.g. mobile telephones) can affect the unit.
- ✚ You must avoid using accessories which are not approved by the manufacturer because it could result in damage to the unit and unsafe use. Also it can lead to an increased emission of electromagnetic waves or to a reduced interference resistance.
- ✚ Don't open the unit. All servicing must be carried out by a company which is authorised by DAVITA®
- ✚ This device must only be used indoors. Do not use in a room which has high humidity. Keep water away from the Control Unit.

Intended usage, effectiveness and treatment plan

- ✚ The DAVITA® Iontophoresis-System SUDORmed DE 20 is used for the treatment of hyperhidrosis (excessive sweating) of the hands, feet and underarms.
- ✚ During the iontophoresis-therapy a low and safe therapy current is conducted through the affected parts of the body. This current flow is enabled because of the water bath and results in decreased sweating of the parts of the body which are in the therapy water.
- ✚ This therapeutic effect has been verified in numerous medical studies. It is not entirely understood how iontophoresis works. Physicians believe that the current flow irritates the synoptical transitions of the sweat

gland nerves so strongly, that they are not stimulated to secrete sweat. This means that the “nervous supply line” is affected and not the sweat gland itself.

- ✚ The strength of the therapeutic current can be controlled to your personal preference, but will not exceed maximum defined values, ensuring safe usage.

The iontophoresis-therapy can be classified into two phases:

- Phase 1: During phase 1, you have to carry out one treatment per day for 20 minutes. After circa 10 – 15 treatments the sweating will be normalized. One treatment per day can be done for a period of 2 – 3 months to reach a permanent relief.
- Phase 2: Successful Iontophoresis therapy requires periodic treatments over the years in order to keep sweating normalised. Depending on the severity of the patient’s hyperhidrosis, sweating will continue to be normalised with a further one to three treatments per week.

Components of the Iontophoresis System SUDORmed DE 20

Your DAVITA®-therapy system consists of a control unit integrated in the case and further accessories, which you will get to know in the following chapter.

Control unit of the SUDORmed DE 20 integrated in the case


Accessories included in delivery

- 1 x pair of electrodes with sponge case 135 x 100 mm
- 1 x pair of electrodes with sponge case 90 x 50 mm
- 2 x leads of electrodes
- 4 x rechargeable batteries 1,2 V
- 1 x charging device
- 2 x large trays (LxWxH): 40 x 29 x 14 cm
- 2 x small trays (LxWxH): 26 x 20 x 7 cm

Operation elements of the control unit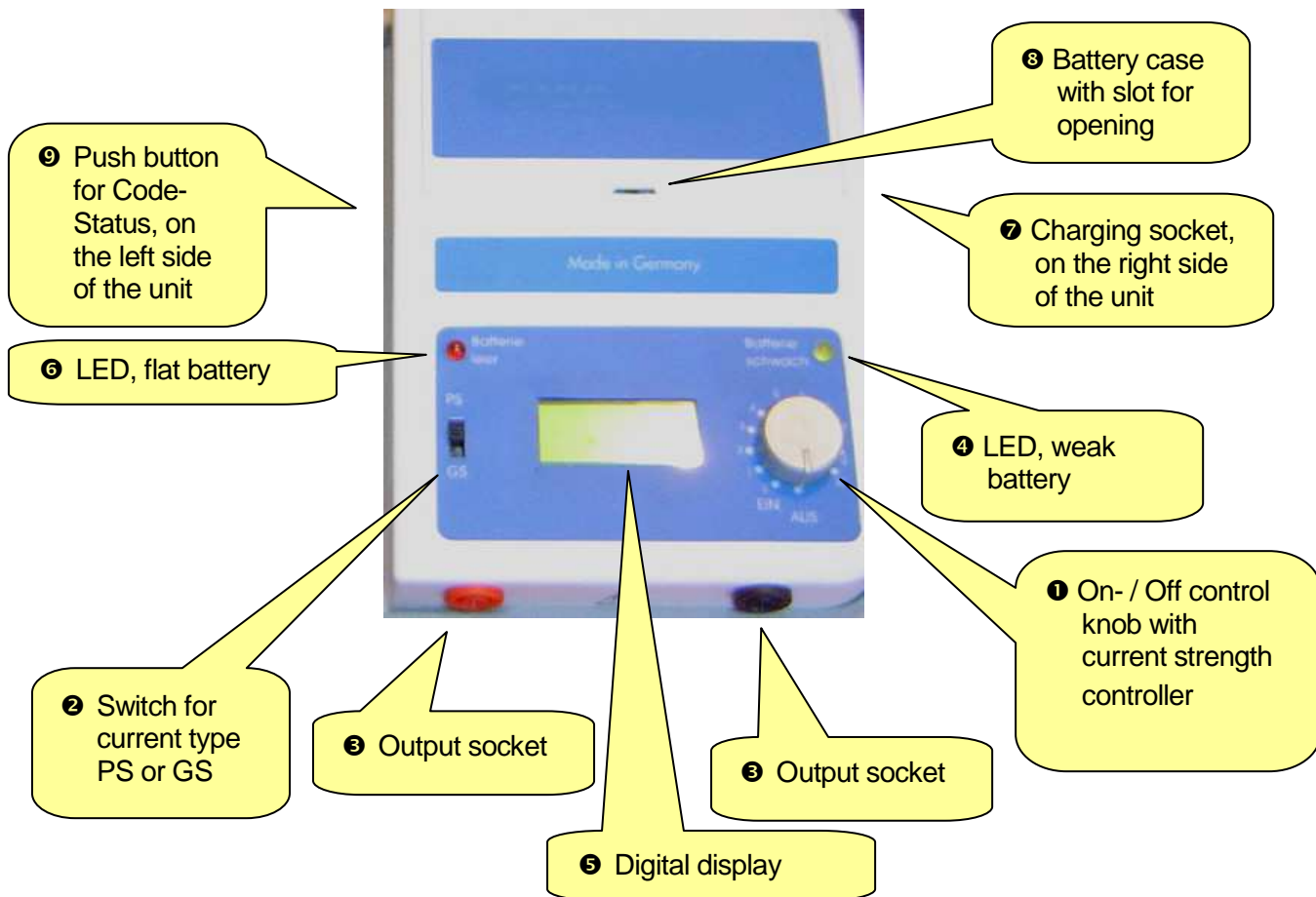

Starting up the Iontophoresis-System SUDORmed DE 20 and therapy performance

Summary of starting up and therapy performance

- 1. Current type:** With this switch 2 you select the current type GS or PS
- 2. On-/Off control knob:** If you turn this control knob 1 to the left side until the point " 0 " appears , you switch on the unit. For switching off the unit you have to turn back the control knob until it is locked again. Thereby you hear a click.
- 3. Digital display 5:** The digital display can show the current type and the Milliampere. Additionally, it shows if the battery voltage has decreased in values, which indicates it is necessary to recharge the battery.
- 4. Current strength controller 1:** After placing the electrodes, the cable connector has to plug into the output sockets. Then the unit has to be switched on and the current strength controller has to be turned on slowly until the right current strength is adjusted according to your personal feeling. The current strength is controlled by turning the knob from the lowest current indicated by "1" to the highest current, indicated by "10".
- 5. Reverse polarity:** The reverse of polarity of the current flow happens by replugging of the cable. The red cable will be inserted into the black socket and the black cable into the red socket.
- 6. LED 4:** As soon as this LED shines, the rechargeable batteries are weak and you will be able to do approx. 2 treatments. Then the rechargeable batteries have to be charged.
- 7. LED 6:** As soon as this red LED begins to shine, the rechargeable batteries are empty. Before the unit can be started again the rechargeable batteries have to be charged.
- 8. Charging socket 7 :** at this socket the charging device will be connected to charge the unit. If the charging device is connected with the unit it is not possible to do a treatment.
- 9. Battery case 3:** to use this unit 4 batteries or rechargeable batteries are necessary. Rechargeable batteries or batteries of the type " LR14 " have to be inserted.

10. Code Status ⑨: the number of treatments which are done completely is stored and can be readout by pressing this push button before switching on the unit.

Starting up and therapy performance in detail

- ✚ Before starting the treatment, all metallic items (e.g. jewellery) must be removed. Insert the 2-mm male connector of the red and black cables in the electrodes and the electrodes in the sponge case. Now you put either one electrode in the sponge case in a plastic tray which is filled with water or the electrodes in a wet sponge case directly on the part of the body which has to be treated. The electrodes (black silicone parts) should not touch the users body. The sponge-covered electrodes should be covered with water for the application.
- ✚ Insert the cables male connectors into each socket of the unit ⑤. When the red cable is connected to the Control unit's red socket and the black cable connected to the black socket it results in the following:- the electrode connected to the red cable being "plus (+)" and the electrode connected to the black cable being "minus (-)".
- ✚ Before switching on the unit select with the switch ② the current type PS pulsating direct current or GS for constant direct current. **The current type GS or PS can only be selected if the unit has been switched off.**
- ✚ Which current type is the better one for you must be decided by the physician or yourself. Constant direct current should be more effective, pulsating direct current is more comfortable for the most patients.
- ✚ To **treat hands or feet** place each sponge-covered electrode separately into the two plastic trays – only one electrode per tray. These non-conductive plastic trays should be filled with luke-warm water. You place the hands or the feet into the trays. For the treatment of hands or feet always use the large sponge-covered electrodes. For the hands use the two small trays and for the feet the two large trays.
- ✚ To **treat onto the underarms or at further parts of the body** put the sponge-covered electrodes directly on the parts of the body which have to be treated. The electrodes must always be covered by the sponges provided. Before placing the electrodes, the sponge cases must be very wet, but not dripping.
- ✚ Turn the control knob ① from "0" to "1" to turn on the unit. The digital display ⑤ will show some text for 5 seconds, indicating that the unit is On. Check that the first row of the display ⑤ matches the switch ② position "GS" or "PS".
- ✚ As soon as the electric circuit is closed, the Milliampere display jumps to the position " 0,4 - 0,7 mA " This value shows that the electric circuit is closed correctly and remains so until the current strength controller is turned up.
- ✚ By turning the control knob ① clockwise you increase the current strength. How much current flows is shown in the second row of the display ⑤.
- ✚ **If the Milliampere display continues to show 0,0 mA, either the skin resistance is too high or the conductance of the water is too low. ¾ fill one of the trays and add common salt and agitate well. The required amount of common salt is individually different and can be up to several teaspoons until the current flow appears. Many users have experienced better results from adding salt.**
We have integrated the measuring of the skin resistance or the conductance of the water, that the current/voltage relation will not become too large.
- ✚ **After 20 minutes the current flow is interrupted automatically.** On the display appears " Behandl. zu Ende "(Treatment is finished). Now you have to turn off the unit, by turning the control knob ① until it "clicks" into its lock position.
- ✚ If the physician has not recommended something else you have to change the polarity for each therapy. The reversing of the polarity occurs if you put the red cable into the black socket and the black cable into the red socket.
- ✚ With the **Code - status** the number of treatments which are done completely is stored. You can readout the Code – status on the digital display ⑤, while you press the push button ⑨ with a pencil which has a diameter of max. 4 mm . This push button you can find on the left side of the unit beside of the label. Then you have to switch on the unit. On the digital display ⑤ you can readout a 4-digit letter combination. If you want to know now how many treatments you have made up to now, you can call us up and we can inform you on the basis of the advised letter combination how many treatments you have done.

Important additional information for therapy performance

- ✚ Because the voltage of constant direct current (GS) is limited on 40 V and for pulsating direct current (PS) to 60 V, the maximum current strength can not be reached if the skin resistance is more than 2000 ohm. Then in the first row of the digital display appears " max ". Now you should try to reduce the skin resistance, while you wet the sponge case again or you have a warm shower. Or you measure the own resistance of the electrodes with an ohmmeter. The electrodes lose their conductivity after some time and must be replaced.
- ✚ If you feel a burning sensation or experience more sensation from one electrode over the other, the electrodes have to be wet again or you have to place the electrodes more precisely to the trigger points.
- ✚ **Too high current strength doesn't lead to better treatment success. On the contrary they can lead to skin irritations. Choose the intensity of the current so that constant direct current (GS) treatments can not be felt and pulsation direct current (PS) treatments can be felt slightly.**
- ✚ **The milliampere reading is a benchmark only because you will feel the current differently for each treatment. The current has to be adjusted according to your personal feeling.**
- ✚ Stop the treatment if skin irritations appear and inform your physician. He will decide how to continue the therapy.
- ✚ It can lead to static discharges if you switch off the unit, whilst the electrodes are still connected. This can be unpleasant but not dangerous. This effect can appear if the ground (for example the carpet) conducts very well. This is only the feeling of static, it is not an electric shock.

Additional information for treatment of hands, feet and underarms

The Iontophoresis-system SUDORmed DE 20 can be applied successfully for the treatment of hands, feet and underarms. Also a simultaneous treatment of the hands and feet is possible.

Treatment of the hands

- ✚ For the first treatment a second person should adjust the current intensity. Make a note of the determined, optimal adjustment, which can be set between 1 and 10.
- ✚ From the second treatment switch on the unit via control knob ❶ and turn the current strength controller up to the determined adjustment. Now you lay the hands in the water filled trays. The current flow is activated as soon you have shut the circuit with your hands. The voltage increases with the speed of 3 V per second until the preset current intensity is reached.
- ✚ You can take out the hands every time and adjust the current intensity with the current strength controller. But the increasing of the current intensity with the control knob ❶ should be done very slowly.

Simultaneous treatment of the hands and feet

- ✚ Feet and hands can be treated simultaneously. You put both feet into a tray and also lay both hands into a tray. To adjust the current intensity you have to put one hand out of the tray and use the current strength controller ❶ to adjust the desired current intensity.
- ✚ After you have adjusted the current intensity optimal, lay the second hand in the tray again. Then all four extremities will be treated simultaneously. A lot of patients have told us, that the therapy is also very successful.

Underarm treatment

- ✚ The fixing of the underarm electrodes is very simple if you wear a narrow shirt. The sponge-covered electrodes have to be pushed under the shirt onto the underarms. The whole sponge cases must be wetted very well, but not dripping. Make sure that the skin of the underarm has a consistent contact with all parts of the sponge-covered electrodes to avoid possible skin irritations.
- ✚

Power supply

Control of battery capacitance

- ✚ As soon as the voltage of the battery decreases to 4,9 volt, the yellow LED ④ shines and on the display ⑤ appears alternating " Batterie laden " (Load battery) and the current type and Milliampere. Now you can do approx. 2 treatments maximal.
- ✚ If the battery voltage decreases under 4,7 volt, the red LED ⑥ shines and shows that the batteries are empty. On the display ⑤ appears " Batterie leer " (battery empty). Now at first you have to recharge the rechargeable batteries or to replace by batteries, before you can continue the treatment.

Charging by inserted rechargeable batteries

- ✚ On the right side of the unit there is a charging socket ⑦. Only the DAVITA® charging device can be use which is concerted upon the integrated rechargeable batteries. If the DC – plug of the charging device has connected with the charging socket, it is not possible to do a treatment because the DC-plug of the charging device will interrupt the electronical circuit from the current supply.
- ✚ **Before you start the unit again, you must pull out the DC – plug of the charging device out of the charging socket.**
- ✚ If there is no possibility to charge the rechargeable batteries (e.g. because you are in a country where the current supply is 110) you can use the unit with 4 batteries (Type LR 14).
- ✚ **It is absolutely forbidden to use a combination of rechargeable and non-rechargeable batteries.**
- ✚ **If the batteries have to be replaced, all 4 batteries have to be changed. We recommend you to use "Alkaline – batteries".**
- ✚ **The charging of empty rechargeable batteries takes approx. 14 hours**

Care, maintenance and disposal

- ✚ Before cleaning the device, remove plugs from power socket and switch off using control knob ①. For cleaning use a soft and wet cloth and a mild cleaning lotion.
- ✚ Don't use petroleum, thinner, alcohol, wax remover or other solvents.
- ✚ Cables must always be removed by holding the connectors. If the cables becomes badly bent or breaks in the insulation appear, then you must replace them. The cables should not be subjected to heat or chemicals.
- ✚ The electrodes and sponges must be cleaned in lukewarm water after every use. Do not use caustic lotions. The sponge cases can be washed in the washing machine together with the hot wash.
- ✚ If you don't use the device for a long time, please take out the rechargeable batteries and store them separately.
- ✚ A special maintenance is not necessary for this device. Only DAVITA® or a company which is authorised from DAVITA® are allowed to do repairs.
- ✚ Packaging: DAVITA® joint to the system „Grüner Punkt“. All parts of the packaging are recyclable materials and environmentally acceptable to dispose. LEDs and units which are signed with the crossed wheelie bin symbol are not allowed to be thrown into the domestic waste. There is the commitment to dispose such electro- and electronic "Altgeräte" (old device) separately. The communal public authority which is responsible for the disposal gives information regarding the possibilities of a regulated disposal.

Guarantee

- ✚ DAVITA® guarantees this instrument for a period of 24 months after date of sale. The guarantee is valid for material defects and manufacturing faults and includes the provision of spare parts as well as labour. Guarantee expires if the instrument has been damaged, or used improperly or unauthorized intervention was carried out. The guarantee becomes invalid if the unit has not been used in accordance with the instructions.
- ✚ Keep the purchase receipt as proof and submit it together with your claim for guarantee. DAVITA® will only fulfil the guarantee upon receipt of your purchase proof.
- ✚ Electrodes , sponge cases, electrode cables and the rechargeable batteries are consumables and excluded from the guarantee.

Technical specifications and scope of delivery

Control unit SUDORmed DE 20

| | | |
|----------------------------------|--------------------|---|
| Dimensions: | 172 x 116 x 41 mm | (W x H x D) |
| Weight: | 270 g | |
| Nominal voltage: | 6,0 V | |
| Constant direct current: | Current intensity: | 0 – 20 mA infinitely variable from 1 to 2000 ohm load resistor max. |
| | Output voltage: | max. 40 V independent from precision resistor |
| Pulsating direct current: | Current intensity: | 0 – 30 mA infinitely variable from 1 to 2000 ohm load resistor max. |
| | Output voltage: | max. 60 V independent from precision resistor |

Iontophoresis-System SUDORmed DE 20, Art. No.: 30034, labelled with CE 0494 consisting of:

- ✚ 1 x Control unit SUDORmed DE 20
- ✚ 1 x Pair of electrodes with sponge case 135 x 100 mm and 1 x Pair of electrodes with sponge case 90 x 50 mm (Art.No.: 35001)
- ✚ 2 x Electrode cables (Art.No.: 35002)
- ✚ 4 x Rechargeable batteries 1,2 V
- ✚ 1 x Charging device
- ✚ 2 x large trays (LxWxH): 40 x 29 x 14 cm
- ✚ 2 x small trays (LxWxH): 26 x 20 x 7cm
- ✚ 1 x detailed and well understandable user manual

CE 0494

Distribution: DAVITA® Medizinische Produkte GmbH & Co. KG
P.O. Box 2004 • D-47518 Kleve
Telephone: +49 (0) 1805 – 911 270 • Fax: +49 (0) 1805 – 911 275
Email: mail@davita.de • www.davita.de

Subject to technical changes. Rev. No. 11010911